CO₂ INHALER – USER MANUAL, QUICK START & SAFETY (ENGLISH, EU VERSION)

Product classification and intended use

The CO₂ Inhaler is a manually operated wellness device intended to deliver air enriched with low concentrations of carbon dioxide (CO₂) from a food-grade source. The device is intended exclusively for general wellness, breathing awareness, relaxation and physiological tolerance training. It is not a medical device under EU Regulation (EU) 2017/745 (MDR) and is not intended to diagnose, treat, cure or prevent disease.

Compatible CO₂ sources

The CO₂ Inhaler is designed for use with certified food-grade CO₂ sources, including SodaStream® cylinders and standard beverage-grade CO₂ tanks fitted with an appropriate regulator. Only food-grade CO₂ may be used. Industrial, technical or non-certified CO₂ must never be connected to the device.

QUICK START

Sit comfortably in an upright position in a well-ventilated room. Place the device on a stable surface. Connect the CO₂ Inhaler securely to a food-grade CO₂ source and attach the breathing mask. Ensure the CO₂ control is set to the lowest level (0–1%) before starting. Hold the mask gently against your face and breathe calmly and normally. Start with a CO₂ concentration of approximately 1–2%. A typical session lasts 5–10 minutes. Sessions may be performed 1–4 times per day depending on individual tolerance. Always increase CO₂ concentration gradually and stop immediately if discomfort occurs.

Interval breathing (optional)

Experienced users may perform interval breathing by briefly increasing the CO₂ level for 10–20 seconds, then lowering it to a more comfortable level until breathing normalizes, and repeating if desired. Avoid prolonged exposure at higher CO₂ concentrations.

SAFETY INFORMATION

Always start at low CO₂ levels. Never force breathing or exceed your personal comfort threshold. Do not perform long sessions at high CO₂ concentrations. Use only while awake, alert and seated. Do not use while driving, operating machinery or performing tasks requiring full attention. Use only in rooms with good ventilation.

System inspection and leak check

Before each use, inspect all connections for leaks, including the connection between the CO₂ tank and regulator, the regulator and hose, and the hose and delivery interface. Leaks can be detected by listening for escaping gas or by applying soapy water to connections and checking for bubbles. Do not use the device if a leak is present.

CO₂ tank handling and storage

Do not store CO₂ cylinders at temperatures above 51.7 °C (125 °F). Excessive heat

increases internal pressure. Always turn off the CO₂ supply when not in use. Ensure the tank neck seal is properly fitted and leak-free.

Ventilation and CO₂ awareness

CO₂ is heavier than air and may accumulate in poorly ventilated areas. Always ensure adequate airflow during and after use. After a session, allow residual CO₂ to dissipate naturally through ventilation.

Cleaning and maintenance

Clean the mask and external surfaces with a damp cloth. Do not immerse the device in water. Do not use solvents or aggressive cleaning agents. Inspect hoses and seals regularly.

User responsibility and disclaimer

The CO₂ Inhaler is provided for general wellness purposes only. No medical claims are made or implied. Uno Vita AS and associated distributors do not provide medical advice and assume no responsibility for improper use or individual outcomes. Use is at the user's own discretion and responsibility. Consult a qualified healthcare professional if you have medical conditions, are pregnant, or are uncertain about suitability.

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